

## IMPORTANT CLEANING INFORMATION - PLEASE READ

### PDT instrument cleaning solution recommendations

The following solutions have been found to be safe for PDT instruments. Please follow all manufacturer guidelines for solution strength and instrument exposure times. All cleaning solutions must be thoroughly rinsed from the instruments prior to autoclaving.

● Product Name	Manufacturer	Destructive
● Sani-treet Green	Enzyme Industries, Inc.	No
● Tri Clean - Triple Enzyme Cleaner	BrandMax	No
● BioSonic Super Concentrate	Coltene/Whaldent	No
● IMS Daily Clean	Hu Friedy	No
● ReSurge Instrument Cleaning Solution	Sultan Healthcare	No
● Alkacide Concentrate	Alkapharm	No
● Alkacide 30	Alkapharm	No
● Neutrasan	Dentisan	No
● Instrument Disinfectant	Unodent	No
● Liquid Detergent	Hospec	No
● NAC Solution	Nusonics	No
● Ultrasonic Cleaner Concentrate	Unodent	No
● Prolystica Neutral	Steris	No
● Dentizyme	Dentisan	No
● Neutral Liquid Detergent	Hospec	No

## IMPORTANT CLEANING INFORMATION - PLEASE READ

### PDT instrument cleaning solution recommendations

The following solutions have been found to be destructive to PDT instruments. Please follow all manufacturer guidelines for solution strength and instrument exposure times. All cleaning solutions must be thoroughly rinsed from the instruments prior to autoclaving.

● Product Name	Manufacturer	Destructive
● Surgical Instrument Cleanser	Unodent	Yes
● pH plus	Dolby Medical	Yes
● Alkazymatic Liquid	Alkapharm	Yes
● Good Vibrations	Certol International	Yes
● Maxicide Plus	Henry Schein	Yes
● VibraKleen E2	Heraeus Kulzer	Yes
● Gigazyme	Schülke & Mayr	Yes
● ProEZ Foam	Certol International	Yes
● Diamond Zyme	Kerr Dental	Yes
● Purit-Cide It	Biotrol	Yes
● Gleam II	Germiphene	Yes
● Ultraclean	Wright Cottrell	Yes
● Instrument Disinfectant	Bossklein	Yes
● Prolystica Alkaline	Steris	Yes
● Dentisan 100	Dentisan	Yes
● Alkazyme	Alkapharm	Yes
● Instrugerm 51	Dento Viractis	Yes
● Instrument & Ultrasonic Cleaner	Continu	Yes